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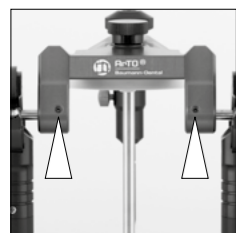
45° Table Support

The 45° degree table support if needed can be inserted into the Articulator base.

Centric Adjustment

If you notice a lateral movement of the condylar axis, while the centric locks are closed, the centric position must be adjusted as below.

Notice: If you had calibrated the articulator it is necessary to recalibrate after repairs.

Loosen the screws by using a 2 mm Allen key.

Adjust the top to center. Pull out the semiaxis till stop and fasten the screws finger tight.

To mount a magnetic plate system, please use the enclosed special screw. Fasten the screw finger-tight by using a 5 mm Allen key. If you need to calibrate the articulator, use the Adesso Split calibration key to glue in the bottom plate. Fixing the base plate only by Adesso Split synchronize adhesive #11030.

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Warning: It is absolutely essential to frequently check calibrated articulators by using a calibration key. Recalibrate the articulator in order to correct discrepancies by using the calibration key.

Cleaning and Care suggestions: Condylar ball and axis path cleanliness maintains reliability and accuracy. Steam clean followed by pressured air is recommended. For more efficient cleaning use the Baumann Dental Cleaning Spray #19088. Spray over area and allow to soak in 1-2 minutes prior steam cleaning. Allow to dry completely. The Baumann Articulator Silicon- Carespray # 11009 is highly recommended to care and protect your Articulator for further longterm usage.


Note on Safety: Your Articulator must never come into contact with acid or any solvents, as this would damage the surfaces and parts. Your Articulator must never be ultrasonically cleaned, this would disable its functional use. In case of neglect or misuse your guarantee rights with Baumann Dental GmbH will void.

Baumann-Dental GmbH
Frankenstr. 25
D-75210 Kelttern
fon +49(0) 7236-933 69 0
info@baumann-dental.de

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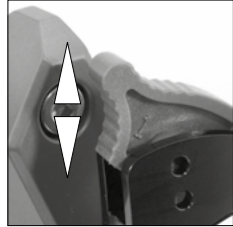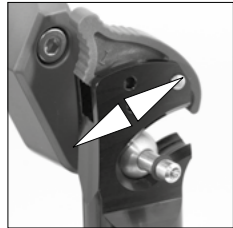
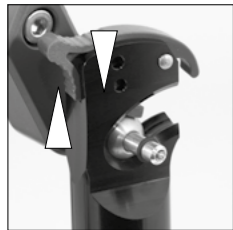
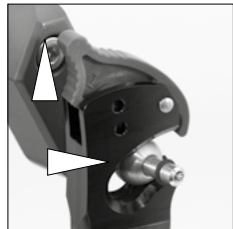
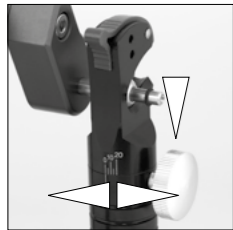
ARTO 1

Centric Locking

Upwards and downwards Locking / Unlocking mechanism.

When the centric lock is unlocked, the upper part of the articulator can move freely in the condylar path. The unlocked lock also prevents the articulator upper part from falling out.

Reposition the condylar ball in it's deepest position within it's path then lock the mechanism rotation movement only may be carried out.

To remove the upper part unlock the mechanism on both sides move the condylar ball within it's path to the highest position for complete removal. By locking the centric locks the upper part is now being released.

Bennet Angle

The Arto 1 Articulator provides a fixed Bennet Angle of 12°.

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ARTO 2

Centric Locking

Upwards and Downwards Locking / Unlocking mechanism.

When the centric lock is unlocked, the upper part of the articulator can move freely in the condylar path. The unlocked lock also prevents the articulator upper part from falling out.

Reposition the condylar ball in it's deepest position within it's path then lock the mechanism rotation movement only may be carried out.

To remove the upper part unlock the mechanism on both sides move the condylar ball within it's path to the highest position for complete removal. By locking the centric locks the upper part is now being released.


Bennet Angle

By loosening the screw the Bennet angle can be variably adjusted. Retighten screw after angle adjustment. The Bennet angle may be adjusted from 0° to 20°.

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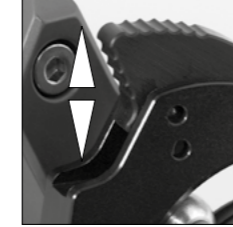ARTO 3

Centric Locking

Upwards and Downwards Locking / Unlocking mechanism.

When the centric lock is unlocked, the upper part of the articulator can move freely in the condylar path. The unlocked lock also prevents the articulator upper part from falling out.

Reposition the condylar ball in it's deepest position within it's path then lock the mechanism rotation movement only may be carried out.

To remove the upper part unlock the mechanism on both sides move the condylar ball within it's path to the highest position for complete removal. By locking the centric locks the upper part is now being released.

Sennet Angle

By loosening the adjustable screw the Bennet angle can be variably adjusted. Retighten adjustment screw after angle adjustment. The Bennetangle may be adjusted from 0° to 20°.

Condylar Inclination Angle

By loosening the condylar adjustment screw the condylar inclination angle can be variably adjusted. Retighten the adjustment screw after angle adjustment. The condylar inclination angle may be adjusted from -15° to +60°.

Arbitrary Axis

Facebows may be attached directly to the articulators Condylars axis pins. Please adjust the Bennett angle to 0° and the Condylar inclination angle to 60°.