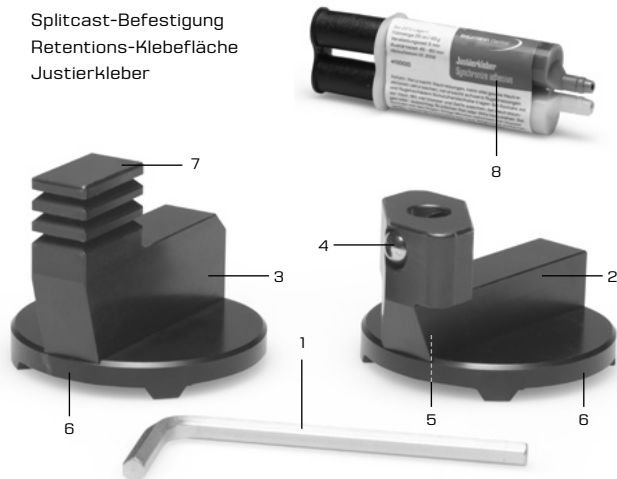
BEDIENUNGSANLEITUNG
STRATOS®-3SHAPE-SCANNER-TRANSFER-SET

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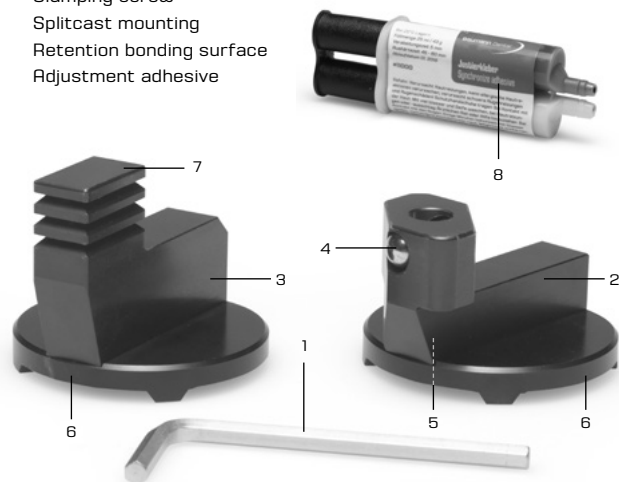
INSTRUCTION MANUAL
STRATOS®-3SHAPE-SCANNER-TRANSFER-SET

PARTS LIST

- 1 Allen key
- 2 Maxillary Splitcast block with clamping device
- 3 Mandibular Splitcast block with bonding device
- 4 Clamping ball
- 5 Clamping screw
- 6 Splitcast mounting
- 7 Retention bonding surface
- 8 Adjustment adhesive


Scope of delivery of the set

- 1 Maxillary Splitcast block with clamping device
- 1 Mandibular Splitcast block with bonding device
- 1 Allen key
- 1 Tube of adjustment adhesive
- 1 Instruction manual

The 3Shape® calibration bodies are not included in the scope of delivery.

BRIEF DESCRIPTION

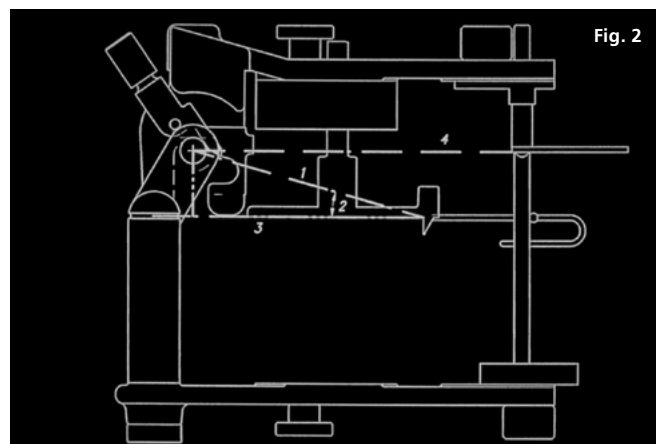Using the Stratos®-3shape® transfer set, the digital data transfer from the Stratos® articulator to the 3shape® scanner can be accomplished easily and precisely. The set can also be used to ensure that the occlusal plane (Fig. 2, point 3), the Bonnwill triangle (point 1) and the Balkwill angle (point 2) are transferred quickly, precisely and correctly to the 3shape® software. This set also ensures bilateral symmetry and alignment of the calibration elements with respect to the central axis (Fig. 3). This set completely dispenses with complicated and time-consuming plaster work.

After mounting and assembly in the Stratos® articulator, the parts can be removed from the articulator using the Splitcast magnetic plate system and transferred to the 3shape scanner.

The scanning process for calibration in the 3shape® scanner can then be executed (Fig. 4)

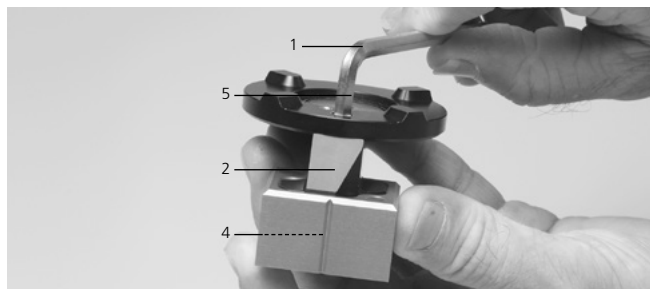
OPERATION

Fastening the Adesso Split magnetic plates in the Stratos articulator. Fasten the Adesso Split magnetic plates (Baumann Dental #10008) in the Stratos® articulator. Set all joint settings to 0 and close the centric fixation of the articulator. The height of the incisal pin must also be set to the neutral 0 position. Optionally, the Stratos® articulator can also be synchronised using the synchronisation set and centering key (Baumann Dental #10025). For further information please visit our website: www.baumann-dental.de

Mounting the maxillary occlusion transfer calibration object. Mount the maxillary part of the occlusion transfer calibration object on the maxillary Splitcast block up to the stop with the clamping device (2). Fixate the clamping screw (5) of the clamping device with the Allen key (1), whereby the clamping ball (4) is pressed onto the clamping surface. Only hand-tighten the clamping screw to avoid damaging the clamping device.


Place the maxillary Splitcast block on the maxillary Adesso Split magnetic plate

Now place the maxillary Splitcast block with clamping device (2) with the magnet connection (6) on the maxillary Adesso Split magnet plate in the articulator.

Placing the mandibular Splitcast base on the mandibular Adesso Split magnet plate

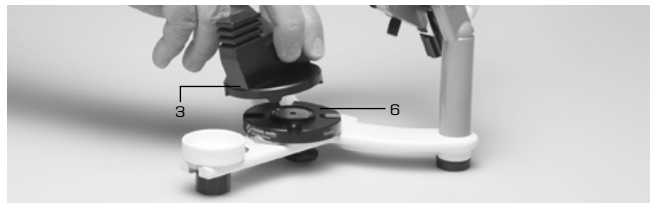Now place the mandibular Splitcast block with clamping device (3) with the magnet connection (6) on the mandibular Adesso Split magnet plate in the articulator.


Turning the Stratos® articulator by 180° and upside down

Turn the Stratos® articulator by 180° and turn the device upside down as shown in the illustration.

Placing the mandibular occlusion transfer calibration object

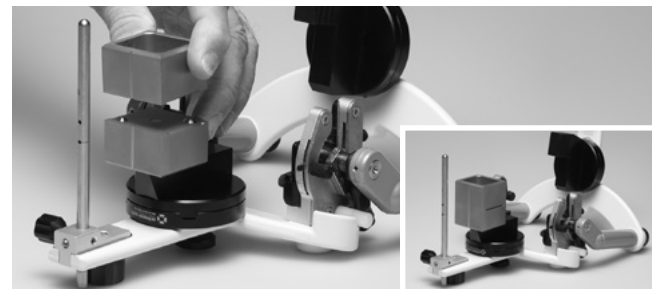Now place the mandibular occlusion transfer calibration object on the maxillary part. The magnetic connection automatically stabilises the part.

Mixing and applying adjustment adhesive

Mix the adjustment adhesive (Baumann Dental #11030) according to the processing instructions and fill the mandibular occlusion transfer calibration object to approximately ¼ of the cavity.

Closing the articulator

Close the articulator and dip the retention adhesive surface (7) into the liquid adjustment adhesive up to the incisal plate stop (please remove excess immediately).

Placing a weight on the articulator and allowing the adjustment adhesive to cure

Place a weight on the articulator to ensure that the articulator remains precisely closed during curing of the adjustment adhesive. Curing time approx. 90-120 min.

Removing the weight and turning the Stratos® over

Now remove the weight and turn the articulator by 180° into the normal position. The process is now completed and the parts can be transferred to the scanner.

Please observe the further instructions of the 3shape scanner.

* Stratos is a registered trademark of Ivoclar Vivadent AG, FL-9494 Schaan, LI
* 3Shape is a registered trademark of 3Shape A/S, DK-1060 Copenhagen K, DK

This device was developed for use in the dental field. Installation and operation must be performed in accordance with the instruction manual. The manufacturer accepts no liability for damages resulting from other use or improper handling. In addition, the user is obliged to check the device before use for suitability and usability for the intended purposes on his own responsibility, especially if these purposes are not listed in the instruction manual.