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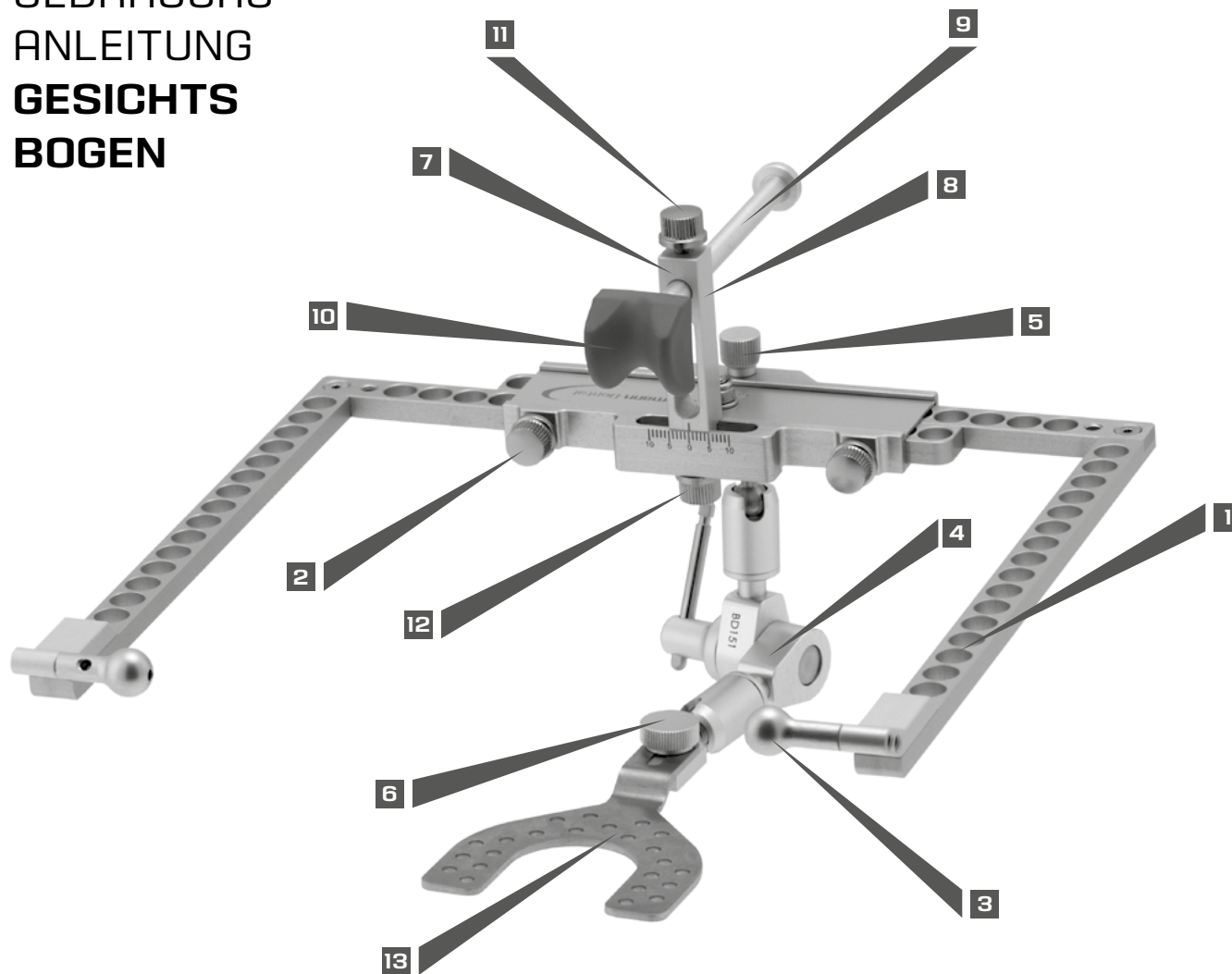
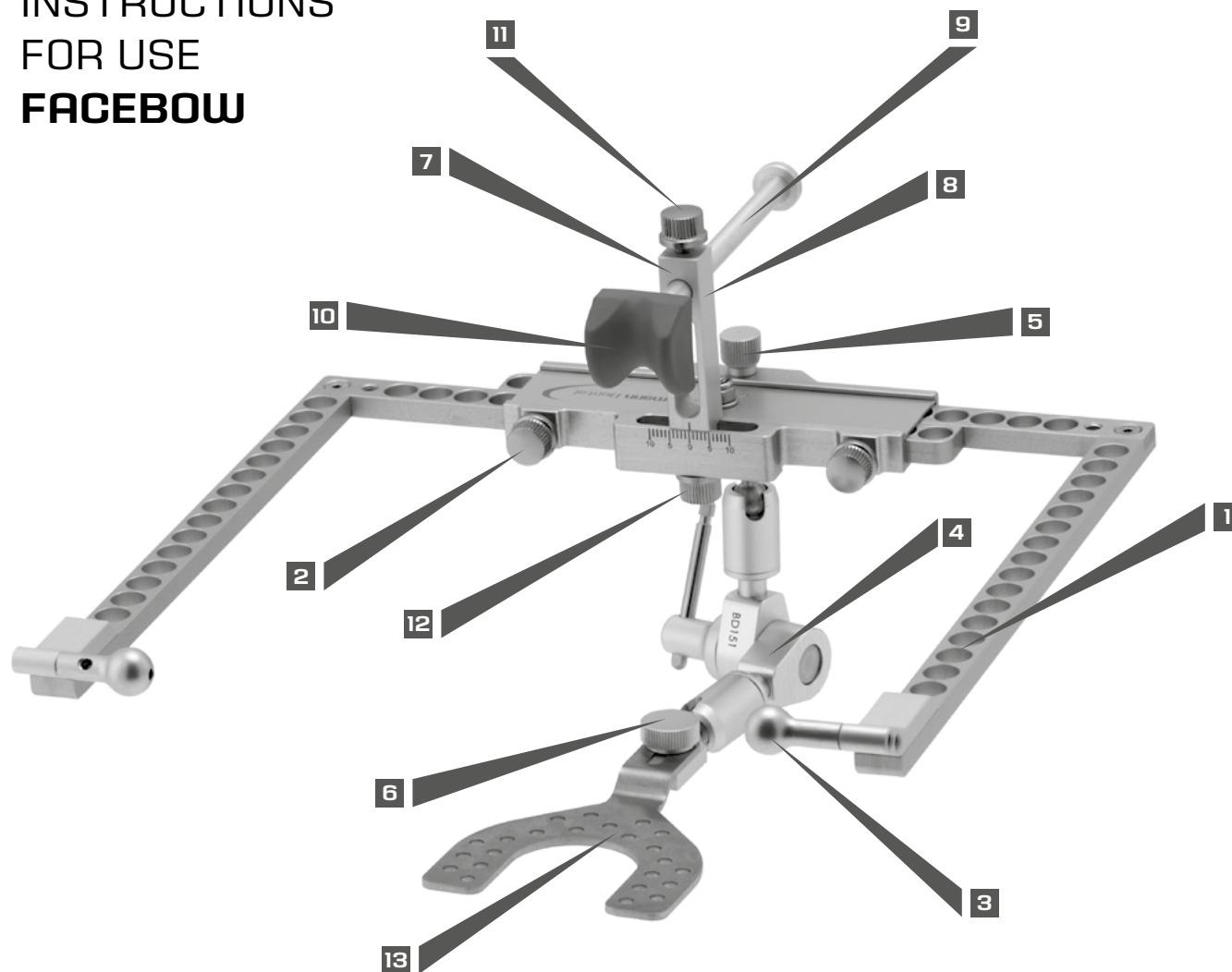
INSTRUCTIONS FOR USE FACEBOW

ENGLISH

Warning: Under no circumstances should the components of the face-bow be cleaned in an ultrasonic bath, as this destroys the anodised coating. Please make sure you follow the instructions under the heading Cleaning and Care on the back.

Face-bow set

71200	Complete face-bow (Face-bow, joint support, glabella support, bite fork x 2)
-------	--

Face-bow

1	71201	Face-bow (bow only)
2	71002	Fixing screw Face-bow
3	71004	Ear pads (x 2)
	71005	Long ear pads (x 2), no dia.
	71006	Axis pin (x 2)

Joint support

4	71110	Complete joint support
5	71111	Support fixing screw
6	71112	Bite fork fixing screw



Glabella support

7	71202	Complete glabella support
8	71203	Glabella bridge holder
9	71003	Glabella bridge
10	71001	Nasion adapter
11	71002	Fixing screw / Glabella bridge
12	71002	Fixing screw / Glabella bridge holder

Bite fork

13	71007	Bite fork (x 2)
----	-------	-----------------

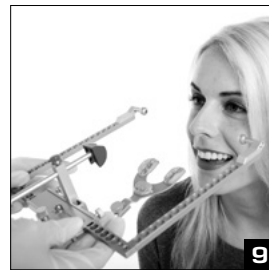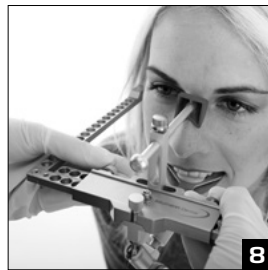INSTRUCTIONS FOR USE FOR THE ARTO FACE-BOW

1 // Pre-assemble as much of the face-bow as possible. Push the nose bridge backwards and pull lightly with the fixing screw. Make sure the joint support's toggle screw is pointing to the right, then place it firmly on the face-bow and pull gently. Unscrew the fixing screws on the face-bow, and open the bow by pulling it apart.

Tip: Only open the toggle screw as far as is possible so that the support can still be moved easily. A quarter turn is usually sufficient for this. This way, the toggle screw will grip straight away during fixing.

2 // Cover the bite fork with a substance such as thermoplastic in at least 3 places. If necessary, the thermoplastic or the prepared bite fork can be warmed up in advance. Press the bite fork against the teeth of the upper jaw so that light impressions are made in the thermoplastic.

Warning: The teeth must not touch the metal of the bite fork!

3 // The bite fork can be placed in the mouth by placing cotton wool underneath it near the molars, or with the help of the patient.

4 // Place the pre-assembled, opened face-bow onto the patient. Place the ear pads of the face-bow onto the patient's external ear canals, and carefully push the bow together while lightly pulling on the bow's fixing screws. Place the glabella support onto the bridge of the nose, and fix it there.

5 // Place the glabella support with built-in lateral adjustment on the patient's median plane, and fix the fixing screw.

6 // Allow joint support to move freely by undoing the toggle screw. While the patient is fixing the bite fork, push the bite fork adapter into the groove of the bite fork. Then screw the fixing screw of the bite fork adapter tight. Now screw the toggle screw tight.

Warning: In order to ensure that the registration can be put into the transfer stand without any problems later, the toggle screw of the joint support must be pointing to the right.

7-8 // Once the impression has been created, undo the bridge screw of the nose bridge and the fixing screws of the face-bow.

9 // After undoing the fixing screws, pull the face-bow and registration gently apart and take carefully off the patient's face.

10 // Impression with transfer stand and transfer table: Remove the joint support from the face-bow and place it into the transfer stand. Screw the joint support's fixing screw tightly. Apply a small quantity of articulation plaster to the plate of the transfer table. Place the transfer table on the transfer stand, and place it right underneath the bite fork until the bite fork has left an impression in the plaster. Now, screw the hexagonal bolt of the transfer table tight.

Tip: It is generally recommended that the registration be impressed in plaster in order to prevent subsequent unintentional manipulation of the registration.

Safety information:

This face-bow is a precision instrument that may only be used by trained dentistry staff. The product may only be used on skin or mucous membranes that have not been injured. We accept no liability for improper use.

Warning:

Please take care when using the bite fork. Introducing it too deeply into the oral cavity or in an improper way can lead to injury. When placing the face-bow on the patient, please take care to avoid injury to the patient in the area of the ear canals.

Cleaning and Care

For all parts of the face-bow and the accessories, in general please use only alkali-free disinfectants and detergents that are suitable for light metals!

Ear pads, nose bridge, nasion adapter
After disinfecting, rinse under running water in order to remove traces of disinfectant.

Preparing the bite fork

Use a spray disinfectant or a disinfectant bath. Sterilisation in the autoclave is not a problem.

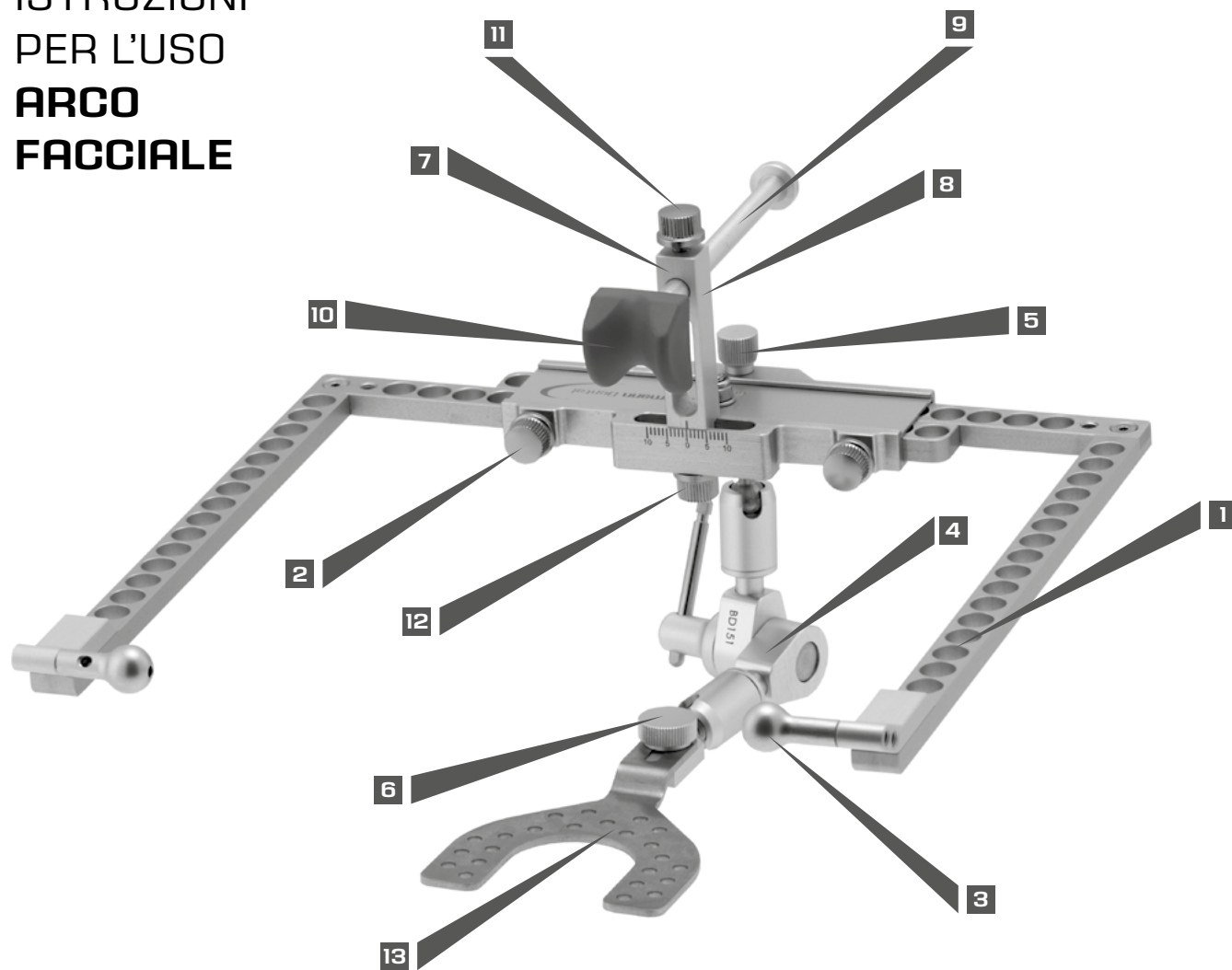
Baumann-Dental GmbH
Frankenstr. 25
D-75210 Kelttern - Germany
Telephone +497236-933690
www.baumann-dental.de

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